



A Brand Division of American Tooth Industries


1200 Stellar Drive • Oxnard, CA 93033-2404
(805) 487-9868 • Toll Free (800) 235-4639 • FAX: (805) 483-8482
E-mail: info@americantooth.com • Web Site: www.americantooth.com



JUSTI-KURA CURING POT

The Justi-Kura™ Curing Pot has been designed to satisfy the need of the modern technician. It is sturdy, very practical, absolutely safe and has a large capacity of 6 quarts. The assembling of the pot is done under strict quality control. The price is very competitive considering that it is made entirely of stainless steel. All parts are available individually.

The Justi-Kura™ Curing Pot is designed for curing Pozzi Ortho Resin, Justi Ortho Resin, and Justi Repair Material. This system ensures a pourous-free cure. When powder and liquid application is complete, cover appliance with warm water in the pot, move the lid to the closed position and inject air 20 to 30 psi. Allow the resin to cure 15 minutes. Exhaust the air by pressing the pressure release valve until the safety lock drops completely. Open lid, remove appliance and allow to bench cool.

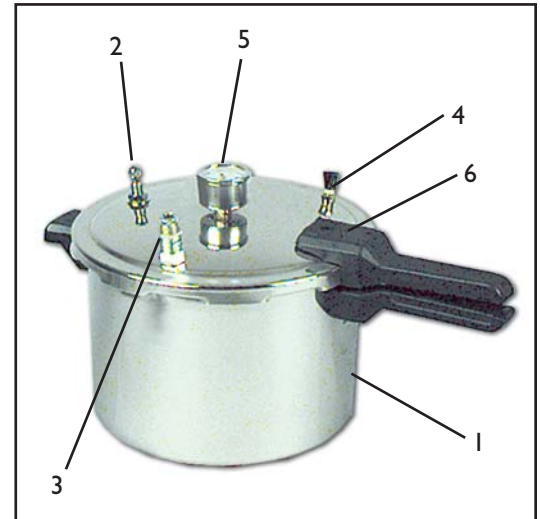

CHARACTERISTICS & ACCESSORIES

Table with 3 columns: No., Description, and Grp/Item #. It lists 7 items including the curing pot, air intake valve, safety relief valve, pressure release valve, air pressure gauge, safety lock, and air supply hose.

OTHER RELIABLE JUSTI PRODUCTS:

Hardened Plastic Teeth:

- Justi Imperial® Anterior & Posterior
Justi Blend® Anterior & Posterior
Dymon-Hue® HPT Anterior & Posterior
Miracleflask Curing System For Dentures, Partials & Relines
Justi Ortho Resin

- Justi Imperial Plastic Facings
Justi Wax Solvent
Justi Milling and Polishing Cream
Justi Resin Cement
Justi Opaquer
Pyroplast Veneering Material
Namilon Resin for Crown and Bridge
Justi Denture Base Material
Justi Repair Resin
and many more...

READ & PASS ALONG TO

Blank lines for writing a name or signature.

Then file for reference

Additional questions, to place your order, or for the Dealer nearest you, please call the Tooth Express Order Desk at 800-628-1437.