



A Brand Division of American Tooth Industries

1200 Stellar Drive • Oxnard, CA 93033-2404
(805) 487-9868 • Toll Free (800) 235-4639 • FAX: (805) 483-8482
E-mail: info@americantooth.com • Web Site: www.americantooth.com



Justi Repair Material

Justi Repair Material is a universal type repair material that is applicable for repairing all types of acrylic dentures, temporary prostheses and indirect relines. A self curing methyl methacrylate material that is cross-linked and internally pigmented to give a dense, even texture.

MIXING & HANDLING:

A mixture of 3 parts powder to 1 part of liquid is recommended for pour or packing technique. For attaching clasp or repairing broken appliances, the powder and liquid alternating technique may be used. Always use a separating medium such as tin foil or liquid tin foil on the model before applying the resin. All types of pressure pot techniques can be used. This will ensure a dense, porous-free repair.

Working time from initial mix is approximately 30 minutes; curing time is approximately 10 minutes in open air at normal room temperature.

TYPE OF PACKAGING:

Justi Repair Material is available in three shades of powder. Regular Pink, Regular Fibered Pink and Clear.



Group/Item # 072-06-0001 - Repair Material Kit

Table with 2 columns: GROUP/ITEM # and DESCRIPTION. Includes REPAIR MATERIAL KIT (20 units) and 1 Ib. powder and 8 oz. liquid sub-sections.

SHADE GUIDE

Table with 2 columns: GROUP/ITEM # and DESCRIPTION. Includes Repair Material Shade Guide.

REFILLS - 1 Ib. powder

Table with 2 columns: GROUP/ITEM # and DESCRIPTION. Lists 1 Ib. powder refills in Regular Pink, Regular Fibered Pink, and Clear.

REFILLS - 7 Ib. bulk powder

Table with 2 columns: GROUP/ITEM # and DESCRIPTION. Lists 7 Ib. bulk powder refills in Regular pink, Regular fibered pink, and Clear.

Table with 2 columns: GROUP/ITEM # and DESCRIPTION. Includes Refills - 25 Ib. bulk powder and Refills - Liquid sub-sections.

NOTE: Keep material away from heat and light to maintain a long shelf life.

\*Enough material for 20 dentures

Products for Better Dentistry

Additional questions, to place your order, or for the Dealer nearest you, please call the Tooth Express Order Desk at 800-628-1437