



A Brand of American Tooth Industries

1200 Stellar Drive • Oxnard, CA 93033-2404  
(805) 487-9868 • Toll Free (800) 235-4639 • FAX: (805) 483-8482  
E-mail: info@americantooth.com • Web Site: www.americantooth.com





# MALIBU SPLIT CRIMPABLE ARCHWIRE STOPS

## SPLIT CRIMPABLE ARCHWIRE STOPS

This remarkable, patent pending, non-slip design is about to change what you think about crimpable archwire stops. Once crimped, Malibu Stops remain securely in place due to the patent-pending, etched surface!

Malibu crimpable split archwire stops are designed to work with any type or size of wire, from nickel titanium to stainless steel, .012 to .021 x .025. No longer will you have to maintain an inventory of several different sizes of stops or waste valuable chair time deciding what size to use. Malibu stops are your solution to every situation where a crimple stop is needed.

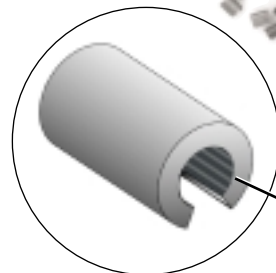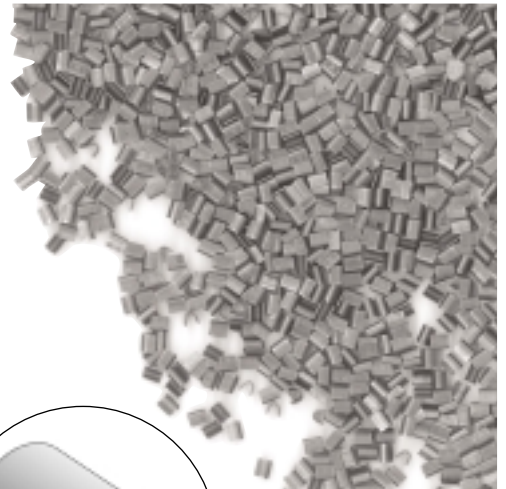

Close-up of split crimpable archwire stop  
Inside view of etched surface

## PLACEMENT IS EASY...


Step One: Place crimpable stop onto archwire using a hard wire cutter

Step Two: Apply firm pressure on the mesial and distal aspects of the stop "crimping" the sides on to the archwire.  
(Crimp firmly 2 times only)

Grp/Item #	Description
031-GCS0500	Malibu Crimpable Stop Pack of 100



Step One: Place crimple stop onto archwire using a hard wire cutter.

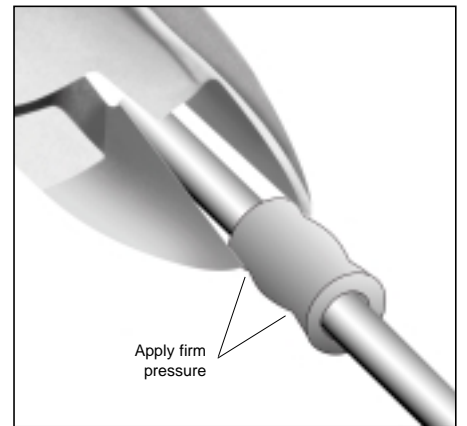

Step Two: Apply firm pressure on the mesial and distal aspects of the stop, "crimping" the sides of the stop onto the archwire.

Additional questions, to place your order, or for the Dealer nearest you, please call the Tooth Express Order Desk at 800-628-1437.