



A Brand Division of American Tooth Industries

1200 Stellar Drive • Oxnard, CA 93033-2404
(805) 487-9868 • Toll Free (800) 235-4639 • FAX: (805) 483-8482
E-mail: info@americantooth.com • Web Site: www.americantooth.com



MILLING AND POLISHING CREAM

An Effective Polishing Agent for Plastic Restorations
An Effective Milling Agent for Plastic Teeth



USES:

- 1. Final occlusal refinement of plastic teeth as a milling media.
2. Removal of residual plaster from cervical and interproximal areas of all plastic dentures.
3. Finishing of cast metal restorations.
4. Removal of adhering plaster from tissue surface side of plastic dentures.
5. Chairside polishing of any plastic restoration.
6. Finishing of amalgam fillings and gold inlays in the mouth.

DIRECTIONS:

POLISHING PLASTIC

Apply cream to area and use with a stiff bristled brush at medium speed.

POLISHING METAL

Apply cream and use high speed with rubber wheel or Abbott brush.

PLASTER REMOVAL

Apply cream liberally to area and use a stiff bristle brush at slow speed. Justi Milling and Polishing Cream is especially useful in removing small bits of adhering plaster investment. The abrasive readily removes the plaster while the lubricant prevents the frictional heat from "burning" the plastic. Remove the Milling and Polishing Cream with Justi Wax Solvent #20 or with a detergent and warm water.

MILLING OCCLUSION

Apply cream to occlusal surfaces and have patient chew in normal excursions. Justi Milling and Polishing Cream will remain on teeth throughout the procedure. It has a pleasant taste and contains no harmful ingredients. Rinse well with warm water to remove residue from mouth.

PACKAGING:

Grp/Item # 070-02-0603 - 1 Milling & Polishing Cream 3 oz.
Grp/Item # 070-02-0604 - 1 Milling & Polishing Cream 1 oz.

Other Reliable Justi Products:

- Justi-Pakto
Justi Miracleflask
Abrasives
Pyroplast Veneering Material
Denture Base Material
Repair Material
Justi Resin Cement
Justi Blend Hardened Plastic Teeth
Justi Imperial Hardened Plastic Teeth
Tray Material
Ortho Resin
Wax Solvent No. 20

READ & PASS ALONG TO

Blank lines for signature or initials

THEN FILE FOR REFERENCE

Products for Better Dentistry

Additional questions, to place your order, or for the Dealer nearest you, please call the Tooth Express Order Desk at 800-628-1437.