

A Brand Division of American Tooth Industries

1200 Stellar Drive • Oxnard, CA 93033-2404  
 (805) 487-9868 • Toll Free (800) 235-4639 • FAX: (805) 483-8482  
 E-mail: info@americantooth.com • Web Site: www.americantooth.com



# GALETTI ARTICULATOR

## with Mechanical Fixation of Models

**Precision Made ■ Original Type ■ Patented**

- Solid construction. Made of metal alloy casting.
- Easy to use, quick fixation of model.
- From chairside to patient - immediate adjustment of the occlusal & correct simulation of central relation.
- All anatomical movements possible.
- Correct occlusion with maximum precision.
- Perfect occlusal-articular balance to prevent cuspidal interferences and to obtain a balanced prosthetic appliance.
- For the dentist, orthodontist and oral surgeon, the Galetti Articulator creates an impressive display that is quick and easy to use for case presentations. It can also be used for study models and has many additional applications for partial and full denture work.




GRP/ITEM #	DESCRIPTION
001-19893	Galetti Articulator, Complete
001-20071	Galetti Horizontal Bolt, Lower
001-20072	Galetti Front Model Model Holding Screw
001-20120	Retaining lever, Galetti Art I pair
001-20121	Galetti Articulator Screw #5
001-20122	Galetti Back Model Hold Screw

## SPECIAL FEATURES

- Easy separation of upper and lower components to form two entirely stable working bases.
- Upper anterior ball bearing joint allows the free motion of upper appliance to check occlusal-articulator balance.
- Hinge-axis with bilateral motion capability for anterior, posterior and lateral movements showing working and balancing movements.
- Adjustable vertical angle pin table to raise or lower the bite.

## INSTRUCTIONS

1. Position upper and lower models by loosening and tightening screws A and B.
2. Unscrew nuts C and D to adapt the upper and lower model in centric relation.
3. Push lever E upward to separate the two models.
4. Turn plate F to raise or lower the bite.

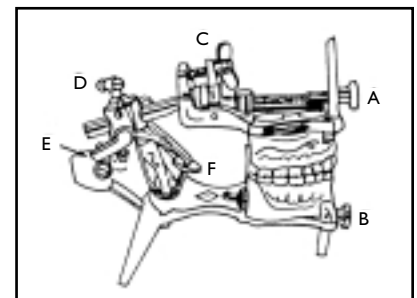

**Additional questions, to place your order, or for the Dealer nearest you, please call the  
Tooth Express Order Desk at 800-628-1437.**

