

A Brand Division of American Tooth Industries

1200 Stellar Drive • Oxnard, CA 93033-2404
(805) 487-9868 • Toll Free (800) 235-4639 • FAX: (805) 483-8482
E-mail: info@americantooth.com • Web Site: www.americantooth.com

The Pozzi Water Flask is made of Heavy Duty Cast Aluminum and is designed to withstand much higher pressures than traditional pressure pots.

Pozzi Water Flask

The Pozzi Water Flask is a precision, economical curing chamber ideally designed for the polymerization of denture repair and orthodontic self curing resins that require polymerization under pressure. This technique is the standard for producing dense, accurate, porosity free acrylic appliances or repairs. Instructions for use:

- *Completely fill the flask with hot water (approx. 138°F - 59°C). Make sure to add enough water that application of the lid causes some to overflow. The Water Flask will not function properly if not completely filled.*

- *Once the appliance to be polymerized is in the water filled chamber and lid secured in place, put the flask into the press and tighten firmly. You will see the pressure increase on the manometer pressure gauge. Tighten until the pressure holds at 5 on the bar scale (72 PSI).*

- *Do not place the flask on flame or heating element during polymerization to avoid damage to the seal and gauge. Allow adequate polymerization time according to your resin manufacturer's recommendation.*

- *Release the pressure completely from the chamber by pressing the release button. To avoid damaging the seal, do not pry the lid open. Caution: Do not exceed 100 PSI.*



Components:

1. Pressure Gauge
2. Pressure Release Valve
3. Thermometer
(Large Flask Only)

Technical Characteristics:

	Small Flask	Large Flask
• Inside dimensions:	100 x 88 mm	100 x 115 mm
• Outside dimensions:	111 x 105 mm	111 x 130 mm
• Made of cast aluminum - industrial finish		
• Incorporated thermometer 20° - 60°C (68°F - 140°F) for denture repairs or orthodontic appliances - (large flask only)		

Guarantee: 3 months

Item No. Description

019-1310	Pozzi Water Flask - Medium*
019-1311	Pozzi Press for Compression Chamber & Water Flask
019-1315	Pozzi Water Flask - Large*

* No air intake valve and no pressure safety valve.

Products for Better Dentistry

Additional questions, to place your order, or for the Dealer nearest you, please call the Tooth Express Order Desk at 800-628-1437.