



A Brand Division of American Tooth Industries

1200 Stellar Drive • Oxnard, CA 93033-2404
(805) 487-9868 • Toll Free (800) 235-4639 • FAX: (805) 483-8482
E-mail: info@americantooth.com • Web Site: www.americantooth.com



VIPI™ FACINGS
A REVOLUTION IN
THE FABRICATION OF TEMPORARY ACRYLIC CROWNS!

FAST! EASY!
TOP QUALITY!



Table with 2 columns: ITEM # and DESCRIPTION. Lists items 67-0010 through 67-0014 and their corresponding descriptions.

VIPI FACINGS make the fabrication of temporary acrylic crowns faster and easier than ever before!

Produced using heat-curing resin of the highest quality, Vipi Facings feature superior strength and durability resulting in longer-lasting temporaries.

Available in 5 Nu-Hue/Classic Molds: A26, 2D, 3M, 3N & 38. One Mold Per Kit.


Each Kit contains 2 facings in each of 3 shades: 62, 66 & 69, for FDI sizes 11, 12, 13, 21, 22, and 23 (maxillary centrals, laterals and first bicuspids) for a total of 36 facings per kit.

INSTRUCTIONS FOR DIRECT USE:

- 1. After preparing the teeth, choose the correct facing, according to the size and shade of the patient's teeth.
2. Modify the shape of the facing to fit perfectly using appropriate abrasives.
3. Using a dapping dish, prepare a self-curing resin of the appropriate shade, and apply it to the lingual side of the facing.
4. Position the facing on the lubricated preparation once the resin has become rubbery, remove the excess.
5. Place the temporary in position and allow the resin to harden.
6. Remove the crown and make any final adjustments. Position the crown and affix using Justi Resin Cement

INSTRUCTIONS FOR LABORATORY USE:

- 1. Prepare the model by exposing and marking the margins. Seal the model using a sealant of choice. Select an appropriate facing according to shape and size.
2. Modify the shape of the facing and adapt it to the models using abrasives/burs.
3. Apply bonding agents and separators as required.
4. Place the facing into position and cure and/or stain using light cure, heat cure, or cold cure systems.
5. Remove the completed crown and make final adjustments. Polish as required.

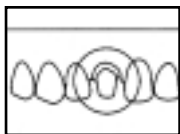

1. Facing selection - Mold and Shade



2. Adjustment Proof



3. Placing the acrylic resin on the facing



4. Frontal positioning of the facing.



5. Lateral positioning of the facing.



6. Final adjustment for cementation.

Additional questions, to place your order, or for the Dealer nearest you, please call the Tooth Express Order Desk at 800-628-1437.