



A Brand Division of American Tooth Industries

1200 Stellar Drive • Oxnard, CA 93033-2404
(805) 487-9868 • Toll Free (800) 235-4639 • FAX: (805) 483-8482
E-mail: info@americantooth.com • Web Site: www.americantooth.com



WAX SOLVENT NO. 20

For wax elimination and wax dressing of acrylic cases



TO REMOVE WAX FILM ON TEETH... Use cotton saturated with Justi Wax Solvent #20, wiping from tooth toward wax-up. Rotate cotton ball. Use a surface with each stroke.

TO DRESS FINISHED WAX UP... Polish with cotton saturated with Justi Wax Solvent #20. Use in mold after boil-out. Work a brush into confining areas of exposed teeth. Flush with clean boiling water. Stand flask on edge to dry.

NOTE... Plastic teeth require complete removal of wax to assure bonding. Justi Wax Solvent #20 may be used with absolute safety in any phase of acrylic processing procedures.

Laboratories report that they are using Justi Wax Solvent also in the following procedures:

1. To add to boil tank water.
2. To clean counter tops.
3. To clean tools and equipment.
4. To remove wax from clothing.
5. To help prevent wax build-up in plumbing.

READ & PASS ALONG TO

THEN FILE FOR REFERENCE

Only Justi Wax Solvent can do the job!

Used by thousands of laboratories in the United States and Canada.

Additional questions, to place your order, or for the Dealer nearest you, please call the Tooth Express Order Desk at 800-628-1437.