



A Brand Division of American Tooth Industries

1200 Stellar Drive | Oxnard, CA 93033-2404

(805) 487-9868 | FAX: (805) 483-8482

info@americantooth.com | americantooth.com

Instruction Sheet

Justi Pressure Pot

Sealing Ring Replacement Procedure

INSTALLATION:

1. Remove the old sealing ring.
2. Clean the sealing ring groove thoroughly. Be sure it is free of all particles and foreign material.
3. **Wash the new sealing ring** in warm soapy water, rinse and dry. Install the sealing ring by working it into the cover a small section at a time until it is fully in place. Any excess amount of sealing ring can be worked into the cove in this manner.
4. If necessary, to help make the cover easier to open and close, a very light coating of vaseline/petroleum jelly may be applied to the underside of the body lugs, **Fig. A. IMPORTANT: NEVER OIL THE SEALING RING.** Oil will swell and soften the sealing ring, reducing the usable life and require frequent replacement.
5. When putting the cover on the body, press one hand down on the area opposite the handle. This will help compress the Sealing Ring and make the cover easy to lock in place. Then open and close the cover several times to make it operate more freely.

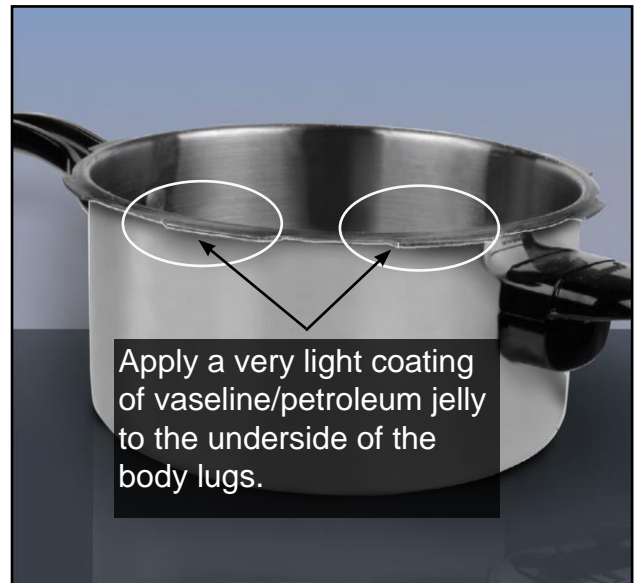


Fig. A. IMPORTANT: NEVER OIL THE SEALING RING.