



A Brand Division of American Tooth Industries

1200 Stellar Drive | Oxnard, CA 93033-2404

(805) 487-9868 | FAX: (805) 483-8482

info@americantooth.com | americantooth.com

Instruction Sheet

Justi Repair Material

Justi Repair Material is a universal type repair material that is applicable for repairing all types of acrylic dentures, temporary prostheses and indirect relines. A self-curing methyl methacrylate material that is cross-linked and internally pigmented to give a dense, even texture.

MIXING & HANDLING:

A mixture of 3 parts powder to 1 part of liquid is recommended for pour or packing technique. For attaching clasp or repairing broken appliances, the powder and liquid alternating technique may be used. Always use a separating medium such as tin foil or liquid tin foil on the model before applying the resin. All types of pressure pot techniques can be used. This will ensure a dense, porous-free repair.

Working time from initial mix is approximately 30 minutes, curing time is approximately 10 minutes in open air at normal room temperature.

TYPE OF PACKAGING:

Justi Repair Material is available in three shades of powder. Regular Pink, Regular Fibered Pink and Clear.



Group/Item # 072-06-0001 – Repair Material Kit

| GROUP/ITEM # | DESCRIPTION |
|--|----------------------|
| REPAIR MATERIAL KIT (20 units)* | |
| 1 lb. powder and 8 oz. liquid | |
| 072-06-0001 | Regular Pink |
| 072-06-0004 | Regular Fibered Pink |
| 072-06-0006 | Clear Powder |

SHADE GUIDE

072-06-0100 Repair Material Shade Guide

REFILLS - 1 lb. powder

| | |
|-------------|----------------------|
| 072-06-0002 | Regular Pink |
| 072-06-0005 | Regular Fibered Pink |
| 072-06-0007 | Clear |

REFILLS - 7 lb. bulk powder

| | |
|-------------|----------------------|
| 072-06-0014 | Regular Pink |
| 072-06-0016 | Regular Fibered Pink |
| 072-06-0017 | Clear |

| GROUP/ITEM # | DESCRIPTION |
|-------------------------------------|----------------------|
| Refills – 25 lb. Bulk Powder | |
| 072-06-0010 | Regular Fibered Pink |
| 072-06-0011 | Regular Pink |
| 072-06-0012 | Clear |

Refills - Liquid

| | |
|-------------|---------------|
| 072-06-0003 | Liquid 8 oz. |
| 072-06-0015 | Liquid 1 qt. |
| 072-06-0013 | Liquid 1 gal. |

NOTE: Keep material away from heat and light to maintain a long shelf life.

*Enough material for 20 dentures